



Cleaning Instructions for Hopkins® Touch Free Thermometer #571500:

Cleaning the Waveguide:

The thermometer waveguide is very delicate, therefore we recommend keeping the protective cap on when it is not in use. If you need to remove dust or dirt from the waveguide, use a cotton swab that has been slightly dampened with alcohol. Remove all dirt and make certain that nothing accumulates at the bottom of the waveguide where the sensor is located. Do not use any other objects or liquids as the surface of the waveguide or the sensor could be easily scratched or damaged. Never let any excess liquid penetrate into the waveguide and sensor.

Cleaning the Thermometer Body:

Use a soft cloth dampened with soap and water. Re-wipe with a sodium hypochlorite disinfectant if desired.

DO NOT USE the thermometer for at least 30 minutes after cleaning.

2016 Hopkins Medical Products