


Mini Infection Prevention PPE Isolation Cart




CT100-0000

LOCKING CASTERS

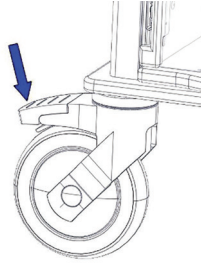

To lock the brake on the casters, apply downward pressure on the foot pedal until it “clicks”. To release the brake, lift the foot pedal back up until it “clicks” into place.

DRAWER SLIDES

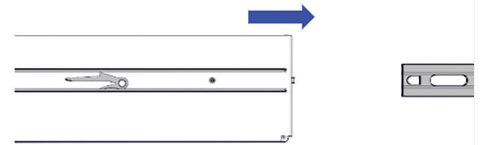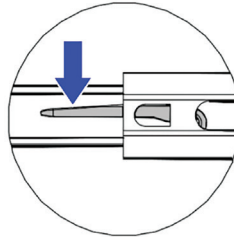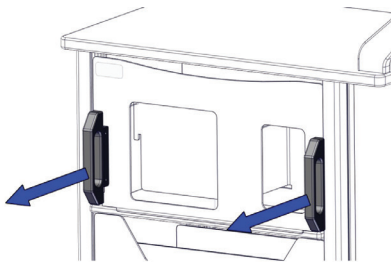


To open a drawer, pull the handle located on the bottom of each drawer.

To remove a drawer, lock the casters and fully open the drawer. Locate the latches located on each drawer slide. Press and hold the latch mechanism on each side of the drawer simultaneously. Gently pull the drawer towards you until the drawer fully disengages from the drawer slide.


To insert the drawer, align the drawer so the slides on each side of the drawer align with the tracks on the cart. Gently push the drawer into the cart until it is fully closed.

SUPPLY LOADING

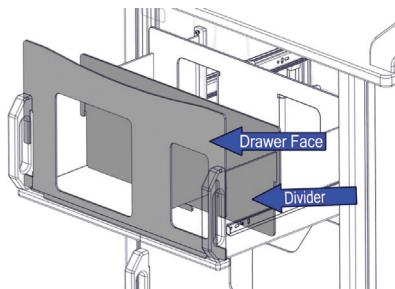

To load PPE for dispensing, open the drawer and place the supplies in the designated area between the divider and the drawer face. Verify that items are properly positioned for dispensing. When finished, close the drawer and the cart is ready to dispense supplies. Extra supplies can be placed behind the dividers.

DRAWER DIVIDERS

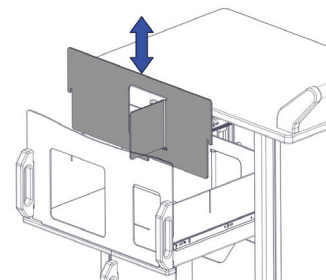

Drawer dividers can be removed by lifting them vertically from their slotted position in the drawer. To replace the divider, align it with the slots in the drawer and slide back into position.

CLEANING INSTRUCTIONS	CAUTION
<p>Cart and plastic components should only be cleaned with soft cloth using a mild soap or germicidal wipes. Alcohol, Windex® or other harsh chemicals will cause plastic to CRACK. Be careful when disinfecting countertops with cleaners. If the overspray accidentally lands on the cart and plastic components it could cause them to crack</p>	<ul style="list-style-type: none"> - Each drawer has a weight limit not to exceed 15 lbs. - Open only one drawer at a time - Lock both casters when stocking the cart with supplies or it is in dispensing mode - Do not sit or stand on the cart - Work surface has a weight limit not to exceed 40 lbs.

CART LAYOUT

ALL DRAWERS HAVE STORAGE BETWEEN THE DISPENSING FRONTS

