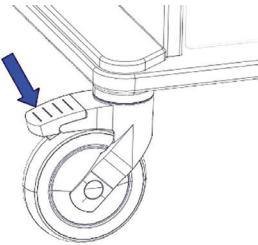
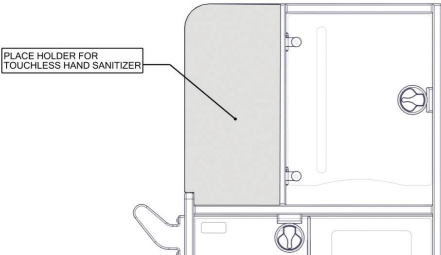
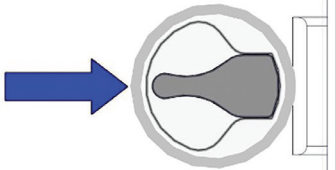
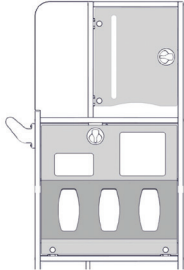
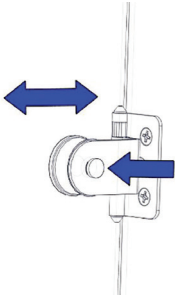
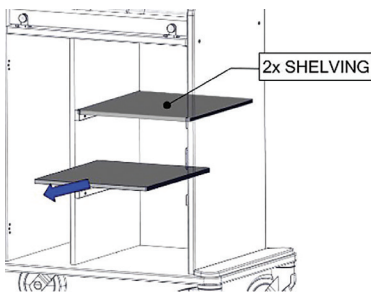


Premium Infection Prevention PPE Isolation Cart

CT300-0000

LOCKING CASTERS	HAND SANITIZER (NOT INCLUDED)
	
<p>To lock the brake on the casters, apply downward pressure on the foot pedal until it “clicks.” To release the brake, lift the foot pedal back up until it “clicks” into place.</p>	<p>A hand sanitizer may be mounted to the open panel on the front of the cart. The hand sanitizer can be mounted by using double sided tape or pan head screws that do not exceed 3/8” in length. The hand sanitizer should be located centrally and near the top edge of the open panel. Leave enough space for any optional drip tray that may be installed later.</p>
DOOR LATCHES	PRODUCT LOAD
	
<p>To open the doors, pull the latch to release the door. To close the door, push the door into the closed position until it “clicks”.</p>	<p>To load PPE for dispensing, open the door and place the supplies in designated areas. Verify that items are properly positioned for dispensing. When finished, close the door and the cart is ready to dispense supplies.</p>
DOOR HINGE ADJUSTMENT	SHELVES
	
<p>To realign the door, use a Phillips head screw driver to loosen the hinge screw and adjust the door (right or left). Once the door is aligned, re-tighten the screw.</p>	<p>The shelves are removable to accommodate various storage needs. To remove, lift the shelf and pull it out of the cart. To replace, slide the shelf back into place and ensure it is seated properly.</p>

CLEANING INSTRUCTIONS	CAUTION
Wipe surfaces with a soft cloth using mild soap and water. Alcolohol, Windex® or other harsh chemicals will cause plastic to CRACK. Be careful when disinfecting countertops with cleaners. If the overspray accidently lands on the cart and plastic components it could cause them to crack.	<ul style="list-style-type: none">- Each compartment has a weight limit not to exceed 15 lbs.- Lock both casters when stocking the cart with supplies or it is in dispensing mode

CART LAYOUT

ALL DRAWERS HAVE STORAGE BETWEEN THE DISPENSING FRONTS

