



## **Insight® Phlebotomy, Storage, Titan, Trans & GynoCart Cleaning & Disinfection Instructions**

### **WARNING!**

- Always comply with facility cleaning standards and guidelines.
- Always follow directions on cleaning product label and heed warnings.
- Before using any cleaner, do a spot test on a small, inconspicuous area of your product to ensure it will not cause damage to the finish.
- Solvents, acidic or abrasive cleaners can damage surfaces and are not recommended.
- Wipe spills away promptly and rinse several times with water.
- Steel wool and other abrasive cleaning pads will damage the surface.
- Do not mix different cleaning agents to clean or disinfect.
- Avoid rust remover, drain cleaners and toilet bowl cleaners. These contain harsh chemicals and will quickly cause permanent damage.
- Avoid hair, textile, and food dyes, as they can cause permanent stains.
- Disinfectants with high concentrations of isopropyl alcohol and / or bleach can be corrosive to plastic with prolonged use.

### **PRODUCT FOR GENERAL CLEANING**

Soap and water are recommended for general cleaning needs.

### **PRODUCT FOR DISINFECTION**

When disinfection is necessary the following products may be used:

- Super Sani-Cloth®
- Sani-Cloth® Bleach
- Sani-Cloth® AF3
- CaviWipes™
- Sani-Cloth® Prime
- CaviWipes™ 2.0
- Sani-Cloth® Plus

### **RECOMMENDED CLEANING PROCESS**

1. Wipe up any spills and remove any debris before applying cleaner.
2. Apply cleaner/disinfectant according to directions on the cleaning agent label.
3. Avoid using sprays on products with electronic components (e.g., electronic locks).
4. Ensure that cleaning solution is wiped away promptly, adhering to the label instructions.
5. Dispose used cleaning cloths and solutions appropriately.