

Assembly Guide

Thank you for choosing us for your phlebotomy storage and organizational needs. You should receive your cart in pristine condition and with all necessary components included. This guide will provide you with a step-by-step walkthrough, to introduce you to the cart's core features and to assist you with its initial setup.

Included Components

- Three pre-installed tilt bins
- Three pre-attached pockets for side rails
- Electronic lock (21491E and 138760E only)

Getting Started

For All Models



To engage the locking casters, apply downward pressure on the white pedal until you hear and feel a "click."

For 21491E and 138760E E-Lock Models Only



1.

For added security this cart is equipped with a digital lock system. Initial code is set as **1-2-3-4**, then press Enter.

a.

To change the lock code, see the lock instructions document included with your cart.

b.

DO NOT THROW AWAY INSTRUCTIONS OR GREEN RESET CODE LABELS.

c.

We recommend keeping track of locks and their respective serial numbers & reset codes.

d.

For any lock issues please reach out to CompX's 24/7 Customer Service 847-752-2525



Scan for Full
Cleaning Instructions

Care Instructions for Your Cart

Soap and water are recommended for general cleaning needs. Solvents, acidic or abrasive cleaners can damage surfaces and are not recommended.

Accessories & Replacement Parts

Accessories and replacement parts are available at marketlab.com or by contacting our Customer Experience Team or your Marketlab Account Manager.



3-Drawer Key Lock
21491



3-Drawer E-Lock
21491E



5-Drawer Key Lock
138760



5-Drawer E-Lock
138760E

MLIFU10252903



Insight[®] Ergo Phlebotomy Cart

Instructions for Use

Item #s: 101878, 101878E, 138758, 138758E, 108687, 108687E, 108689, 108689E

Assembly Guide

Thank you for choosing us for your phlebotomy storage and organizational needs. You should receive your cart in pristine condition and with all necessary components included. This guide will provide you with a step-by-step walkthrough, to introduce you to the cart's core features and to assist you with its initial setup.

Included Components

- Three pre-installed tilt bins
- Three pre-attached pockets for side rails Electronic lock (101878E, 138758E, 108687E, and 108689E only)
- Guard Rail Top (108689 and 108689E only)
- Tracking caster (108687 and 108687E only)

Getting Started

For All Models



To engage the **locking caster**, apply downward pressure on the **white pedal** until you hear and feel a “click.”

For 108687 & 108687E



To engage the directional lock on the **tracking caster**, apply downward pressure on the **gray pedal**. This prevents the caster from swiveling.

For full maneuverability and ability to turn the cart in any direction, disengage the lock by pressing downward a second time.

For 101878E, 138758E, 108687E, 108689E E-Lock Models Only



1.

For added security this cart is equipped with a digital lock system. Initial code is set as **1-2-3-4**, then **press Enter**.

a.

To change the lock code, see the lock instructions document included with your cart.

b.

DO NOT THROW AWAY INSTRUCTIONS OR GREEN RESET CODE LABELS.

c.

We recommend keeping track of locks and their respective serial numbers & reset codes.

d.

For any lock issues please reach out to CompX's 24/7 Customer Service 847-752-2525

Care Instructions for Your Cart

Soap and water are recommended for general cleaning needs. Solvents, acidic or abrasive cleaners can damage surfaces and are not recommended.

Accessories & Replacement Parts

Accessories and replacement parts are available at marketlab.com or by contacting our Customer Experience Team or your Marketlab Account Manager.

Do You Need Assistance?

Please call our Customer Experience Team at 800.237.3604 or email us at service@marketlab.com.



Scan for Full
Cleaning Instructions



**3-Drawer
Key Lock**
101878



**3-Drawer
E-Lock**
101878E



**5-Drawer
Key Lock**
138758



**5-Drawer
E-Lock**
138758E



**5-Drawer
Tracking Caster,
Key Lock**
108687



**5-Drawer
Tracking Caster,
E-Lock**
108687E



**5-Drawer
Guard Rail Top,
Key Lock**
108689



**5-Drawer
Guard Rail Top,
E-Lock**
108689E



Do You Need Assistance?

Please call our Customer Experience Team at 800.237.3604 or email us at service@marketlab.com